

Fit Checker™ Advanced & Fit CheckerTM Advanced Blue



of prosthetic work.





Fit Checker™ Advanced Blue

The ideal blue colour gradation for



- Fit checking of all kinds of prosthetic works, especially resin, ceramic and zirconiabased crowns and bridges.
- Bite registration and evaluation of occlusal contacts.



Balanced thixotropy allows for a non-runny consistency. Under pressure, Fit Checker Advanced Blue offers easy flow and extremely thin film thickness for a perfect fit.



Misfits are easily detected thanks to **optimal transparency** and unique **blue colour gradation**.



Finally, it's possible to make certain of a **perfect fit for indirect restorations**, avoiding marginal gaps and major occlusal adjustments after cementation.

Snap set
Sufficient working time and rapid setting (1 min. in the mouth)
for a fast and comfortable procedure.



2 Clear and precise visualisation
A perfect colour gradation for immediate visualisation
of occlusal contacts and poorly fitting areas.



Fit CheckerTM Advanced

The ideal white colour < gradation for

• Fit checking of all kinds of prosthetic works, especially complete and partial dentures and metal-based crowns and bridges.



The perfect solution for checking the adaptation of complete and partial dentures.



Fit Checker Advanced offers ideal consistency and high wettability for easy flow into denture bases and oral tissues. Fit Checker Advanced is easily removed in one step.



Denture misfits can then be corrected with acrylic reliners. After relining, final checking is performed to assure patient comfort.

No mess during adjustments
The set material does not get caught by the bur while relieving pressure spots.



Easy to peel off
The silicone pellicle peels off easily for a clean and fast procedure.



The simplest way towards successful finalisation of your prosthetic work

Fit Checker Advanced and Fit Checker Advanced Blue from GC provide a simple and precise way to assess the fit of prosthetics and make timely adjustments at the fitting/seating appointment - avoiding potential future costs and ensuring patient comfort.

Fit Checker™ Advanced	Benefit	Fit Checker ™ Advanced Blue
VPES Silicone	Innovative silicone material with intrinsic wettability, flowabilty and resistance	VPES Silicone
White	Two different colours to meet all clinical requirements	Blue
Working time: 1 min. Setting time: 1 min. in mouth	Sufficient working time and a short setting time in the mouth	Working time: 1 min. Setting time: 1 min. in mouth
Handmix (tubes) or Automix (cartridges)	Automix cartridges available for an easy, clean and fast procedure	Automix cartridges







Pac	kaq	es

GC Fit Checker™ Advanced / GC Fit Checker™ Advanced Blue		
004903	Fit Checker Advanced, 1-1 Pack (55 g + 50 g), White + retarder (3g) + accessories	
004904	Fit Checker Advanced, Cartridges (2 x 62 g), White + mixing tips IIS (3) and IISS (3)	
004905	Fit Checker Advanced Blue, Cartridges (2 x 56 g), Blue + mixing tips IIS (3) and IISS (3)	
800006	GC Cartridge Dispenser II	
890130	GC Mixing Tips II, 60 x SS (Yellow)	
800100	GC Mixing Tips II, 60 x S (Pink)	

GC EUROPE N.V.

Head Office Researchpark Haasrode-Leuven 1240 Interleuvenlaan 33 B-3001 Leuven Tel. +32.16.74.10.00 Fax. +32.16.40.48.32 info@gceurope.com http://www.gceurope.com

GC UNITED KINGDOM Ltd.

12-15, Coopers Court Newport Pagnell UK-Bucks. MK16 8JS Tel. +44.1908.218.999 Fax. +44.1908.218.900 info@uk.gceurope.com http://uk.gceurope.com



