

Serious Whitening

A bright smile connects



VivaStyle®

A comprehensive system for esthetic dentistry



Naturally white teeth – a desire frequently voiced by patients.

In order to meet the expectations of patients, Ivoclar Vivadent offers products for high-quality restorations, such as Tetric EvoCeram[®], IPS Empress[®] or

IPS e.max[®], in Bleach shades. For whitening of natural teeth, VivaStyle is the product of choice. Furthermore, the prophy pastes Proxyt[®] help keep teeth bright and white for a long time.

Your benefits

The treatment strategies, which focus on enhancing dental esthetics, are based on your diagnosis and your patients' individual needs. As is the case with restorative materials, VivaStyle is suitable for different requirements. Therefore, various VivaStyle products are available for the whitening of natural teeth.

Benefits for your patients

Professional care and the use of clinically proven materials and products enable you to meet your patients'demand for naturally white teeth.

Interesting!

Tooth Whitening VivaStyle[®]

98% of all patients recommend tooth whitening to their friends. (Lutz et al. 2000) Starting situation

Situation after tooth whitening



After restoration ³



*Restoration with IPS e.max Press veneers 22-21 Dr L. Enggist, J. Seger

Prosthetics


IPS e.max[®], IPS Empress[®], in Bleach shades

Restoration

Tetric EvoCeram[®], IPS Empress[®]Direct, in Bleach shades

Professional Care Proxyt[®], Fluor Protector, Cervitec[®], VivaSens[®]

VivaStyle releases oxygen which bleaches discolouration.

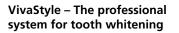

The specialist know-how of the practice team together with a professional bleaching system constitute the main pillars of success. VivaStyle products whiten the teeth, making it easy to provide patients with a bright shiny smile. Individualized demands and situations vary, of course, so that a choice of products for both in–office and at-home use is available.

VivaStyle – The professional decision

In accordance with the diagnosis and individual patient needs, different treatment options are available:

0

0

Ο

- in-office or at-home use
- different concentrations of the active agent
- application:
 - gel with custom-made tray
 - varnish system applied with a brush

Here's how it works

The VivaStyle gel products contain the tried-and-tested ingredient carbamide peroxide in various concentrations and the VivaStyle Paint On Plus varnish system contains hydrogen peroxide. These active ingredients release oxygen which gently bleaches discolouration.



VivaStyle[®] Paint On Plus



VivaStyle[®] 30% VivaStyle[®] 10% | 16%




Which physical feature makes a person attractive? According to 85% of the people interviewed in a survey it is 'beautiful teeth'.

(Survey conducted by Emnid, 2002)

VivaStyle[®] 30 %

For in-office treatment

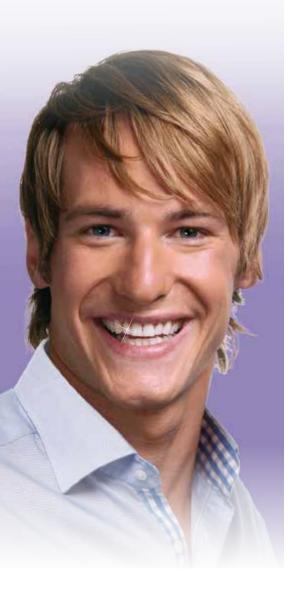

Professional treatment

VivaStyle 30 % * is applied at the dental practice. The gel is applied in a custom-made tray which is then placed in the mouth. The patient can relax in the waiting room whilst the gel takes effect during the next 20 to 30 minutes. The dental chair is free for other treatments during that period.

Advantages

- ▶ potassium nitrate
- rubber dam unnecessary
- 1 to 3 applications

Your benefits

- gentle alternative to highly concentrated hydrogen peroxide products
- faster results with severe discolouration
- alternative for patients who cannot or do not want to use a tray at home

For patients with the following requests

- reduced treatment period
- treatment of severe discolouration
- all-round professional service

* 30 % carbamide peroxide, equalling 10.8 % hydrogen peroxide.

Special tip

Offer patients a motivating start-up treatment at the practice with VivaStyle 30%. A shortened follow-up treatment with VivaStyle 16% can then take place at home.

Interesting!

None of the investigated light sources showed a stronger whitening effect than the sole application of a whitening gel. (CRA Newsletter 2003)







VivaStyle[®] 10 % | 16 %

Gel for treatment at home



Effective whitening using a tray

Treatment with VivaStyle gel containing 10% or 16% carbamide peroxide* takes place at home with a custom-made tray.

Advantages

- smooth viscosity
- potassium nitrate
- calibrated, resealable syringe
- soft, comfortable tray material

Your benefits

The features of VivaStyle enable the following:

- more precise dosage
- prolonged action on the right spot
- less irritation

For patients with the following requests

- professional initial care and subsequent application at home
- gentle whitening
- comfortable application

* 10% /16% carbamide peroxide correspond to 3.6%/5.8% hydrogen peroxide.

Interesting!

During and after a whitening treatment, local fluoridation of the teeth is advisable. (Wiegand & Attin 2002)







Special tip

Tetric EvoCeram[®], IPS Empress[®] or IPS e.max[®] in Bleach shades allow the fabrication of matching functional esthetic restorations.

VivaStyle[®] Paint On Plus

The professional varnish system



Effective whitening without using a tray

VivaStyle Paint On Plus is directly applied to the teeth with a brush. The varnish contains 6 % hydrogen peroxide. After drying, the varnish remains on the teeth for 10 minutes. The varnish system does not dissolve in saliva and takes effect within minutes of being applied. After the treatment, it is simply removed with a toothbrush.

Advantages

- varnish system: for targeted use on the spot
- D-Panthenol: provitamin
- whitening: after just seven days
- in the dental office: intensive treatment

Your benefits

- extension of the services your dental practice offers
- effective whitening without using a tray
- satisfied patients

For patients with the following requests

- professional tooth whitening without using a tray, as they
 - find trays to be uncomfortable
 - are looking for a more cost-effective alternative
- easy to fit into the daily schedule
- gentle application

Special tip

Start treatment at the dental office by professionally cleaning the teeth with the Proxyt[®] prophy paste.



Photos: Prof. Dr Mata and Prof. Dr Marques, Caparica

Intensive treatment

An intensive treatment can be performed in the dental office. In this case, the varnish is applied and allowed to react. Afterwards, the varnish is removed and then applied again. These steps are repeated multiple times during one treatment session (Mata & Marques 2006).

Interesting!

'Paint-On' preparations are useful for whitening discoloured single teeth very selectively. (Wiegand et al. 2006)

VivaStyle[®]

Serious Whitening

VivaStyle – In your professional interest and in the interest of your patients

A smile connects. Recommend the individually suitable VivaStyle product to your patients for a brighter smile.

Comprehensive service Information material for your practice team

- brochures
- flow charts
- specialized articles
- study results
- documentation
- posters



- Information material for
- 'Questions & Answers' brochure, also available with display holder
- flow charts
- posters
- clip for the waiting room



Delivery forms

VivaStyle 30%

Refill Kit

1 syringe VivaStyle 30%, 3 ml

VivaStyle 10 % | VivaStyle 16 %



Patient Kit 10 % | 16 % 4 syringes VivaStyle, 3 ml each 2 sheets tray material 1 tray case Bag

Touch Up Kit 10 % | 16 % 2 syringes VivaStyle, 3 ml each

Refill Tray Sheets 12 sheets tray material

VivaStyle Paint On Plus

Patient Kit

1 tube, 20 ml VivaStyle Paint On Plus 14 brushes 14 dispensing dishes Bag



Quick assessment

Customized treatment

The coordinated Professional Care program simplifies the assessment of the patient's individual treatment needs and enables a successful combination of different treatment options.

CRT[®] bacteria

The test for determing the mutans streptococci and lactobacilli counts in saliva and plaque

CRT[®] buffer The test for determing the buffer capacity of saliva

Plague Test The fluorescent disclosing liquid for detecting plaque

OptraGate® The soft lip and cheek retractor Proxyt[®] The prophy pastes

Cervitec[®]

The protective varnish, oral health care gel and mouth rinse containing chlorhexidine and more

Fluor Protector The protective varnishes and the special care gel containing fluoride and more

VivaSens® The desensitizer for hypersensitive teeth

Literature

Benbachir N, Ardu S, Krejci I: Spectrophotometric evaluation of the efficacy of a new in-office bleaching technique. Quintessence Int 2008; 39: 1-8 Int 2008; 39: 1-8

Calatayud JO, Varga PM, Calatayud CO, Calvo Box MJ: Comparative clinical study of two tooth bleaching protocols with 6 % hydrogen peroxide. Int J Dent 2009; 2009: 928306

Mata ADSP, Marques DNS: A novel technique for in-of-fice bleaching with a 6% hydrogen peroxide paint-on varnish. Eur J Esth Dent 2006; 1: 70-77

Ziebolz D, Hannig C, Attin T: Influence of a desensitizing agent on efficacy of a paint-on bleaching agent. Am J Dent 2008; 21: 77-82

All the studies are available on request.

Fixed Prosthetics

VivaStyle[®] forms a part of the "Fixed Prosthetics" product category. The products of this category cover the procedure involved in the fabrication of fixed prosthetic restorations – from temporization to restoration care. The products are optimally coordinated with each other and enable successful processing and application.



THESE ARE FURTHER PRODUCTS OF THIS CATEGORY:

IPS e.max[®] System

all ceramic - all you need

Multilink[®] Automix

The adhesive cementation system



The comprehensive solution covering all indications

- Highly esthetic, high-strength materials for the press and CAD/CAM technique
- Unique lithium disilicate (LS₂) and zirconium oxide (ZrO₂) ceramics for restorations ranging from thin veneers to long-span bridges
- Flexibility of cementation: adhesive, self-adhesive and conventional



A strong bond, proven performance

- Strong hold both dual and self-curing
- Universal suitable for silicate and oxide ceramics as well as metal
- Clinically proven numerous long-term studies

Would you like to know more about the products of the "Fixed Prosthetics" category? Simply get in touch with your contact person at Ivoclar Vivadent or visit www.ivoclarvivadent.com for more information.

Ivoclar Vivadent AG Bendererstr. 2 9494 Schaan Liechtenstein Tel. +423 235 35 35 Fax +423 235 33 60 www.ivoclarvivadent.com

Descriptions and data constitute no warranty of attributes. Printed in Liechtenstein © Ivoclar Vivadent AG, Schaan/Liechtenstein 665209/en/2013-09-12

