


## SAY HELLO TO OP-D-OP

Experience the seamless integration of style, safety and comfort, while minimizing your contact with splash, splatter and spray.

ALWAYS WEAR YOUR OP-D-OP WITH THE  
FACE SHIELD AND MINI-SHIELD ATTACHED

### THE ABS

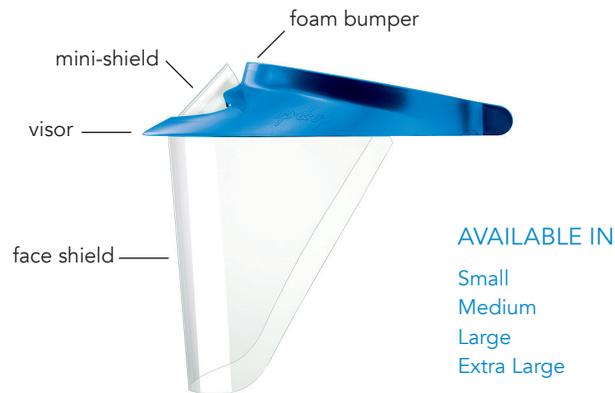

### THE POLYSOFT



## REPLACEMENT SHIELDS AND FOAM

Order replacement face shields, mini-shields and foam bumpers as needed. Your Op-d-op is easy to keep on loving!

## THE OP-D-OP PROMISE

We hope you love your Op-d-op. However, if this isn't the case, let us know so we can make it right. We're here to help.

## LOOK FOR THE REAL DEAL

Look for Op-d-op's trademark punch pattern on your shields and the matching logo shaped pegs inside your visor for proof of Op-d-op authenticity and quality.



## MADE IN CALIFORNIA SOLD AROUND THE WORLD

### Op-d-op, Inc.

10628 Industrial Avenue  
Roseville, CA 95678 USA


T 1.916.783.5741 • F 1.916.783.5765 • E info@opdop.com

[www.opdop.com](http://www.opdop.com)

Op-d-op Face Shields are designed to be worn over the face of health care workers to reduce the risk of being splattered or splashed with possibly infectious bodily fluids. Op-d-op products do not prevent the contraction of disease or infection and should always be worn using standard precautions. Properly disinfect or discard Op-d-op products after each use.




WORLDWIDE PATENTS AND TRADEMARKS APPLY

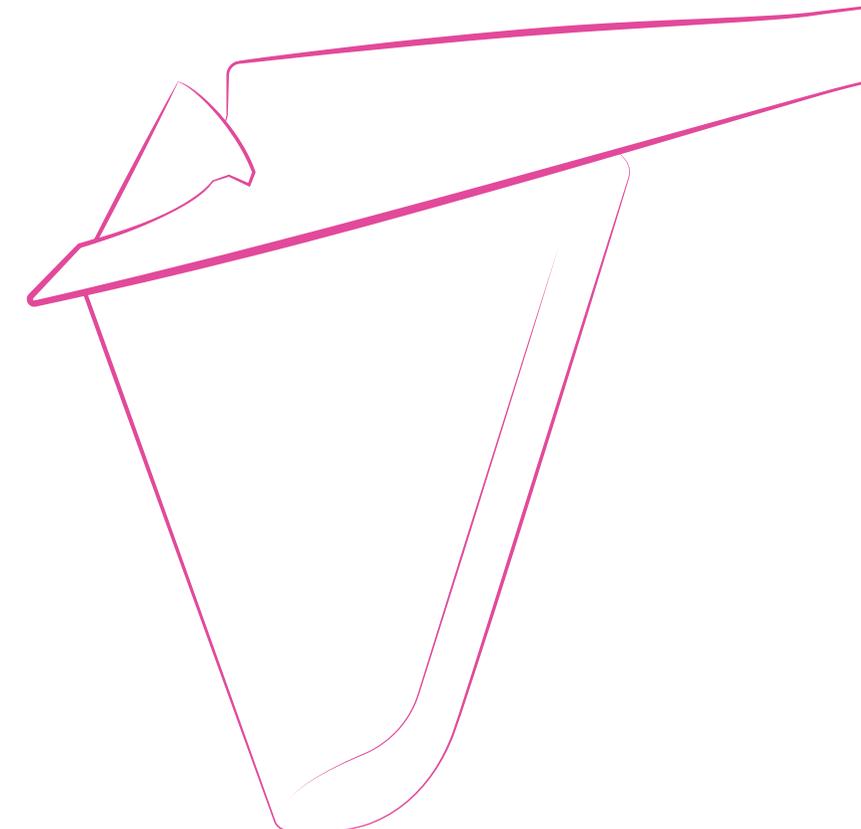


THE ORIGINAL VISOR SHIELD  
Move in Close with Confidence


## 1 REMOVE MINI-SHIELD

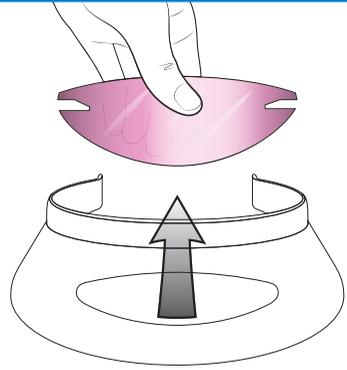


## 2 PEEL SHIELDS

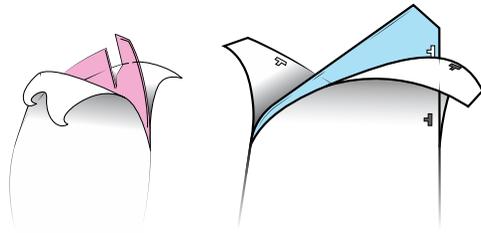

## 3 ATTACH FACE SHIELD

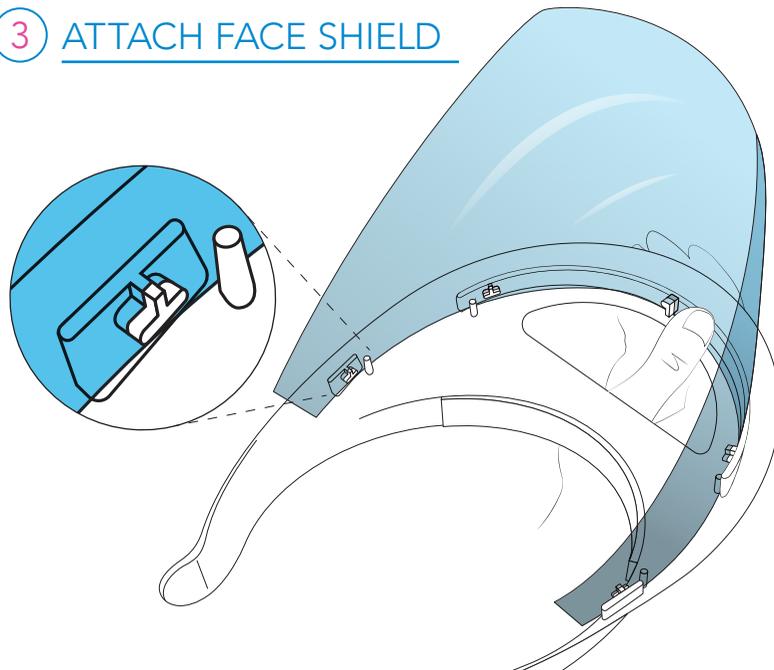


## 4 RE-ATTACH MINI-SHIELD

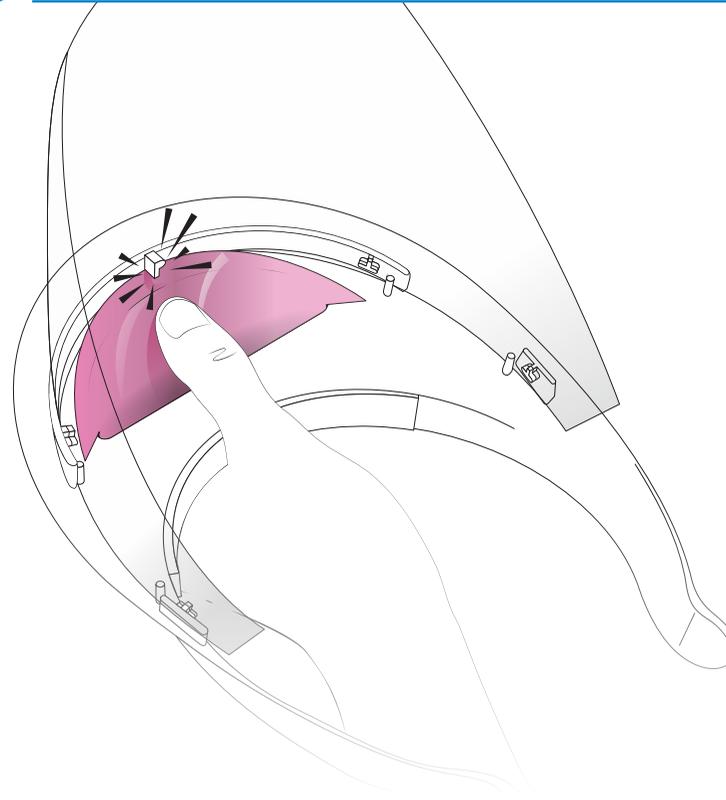

## 5 WEAR IT

To prevent fogging, wear your Op-d-op with the shield parallel to your face - fit two or three fingers between your chin and the shield to check.

You're ready to go!

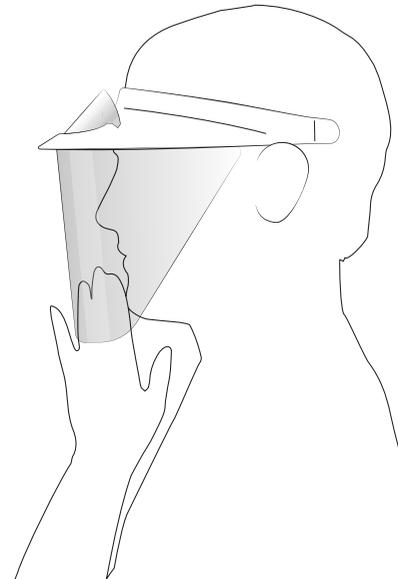

## SHIELDS TOO LONG?

If the shield touches your shoulder or chest when working, shorten the shield with scissors. Just don't cut too much off!

## CLEANING YOUR OP-D-OP

If your shield is heavily splattered, properly dispose of it and replace with a new one.

To reuse shields, wash thoroughly under running water using an antimicrobial soap (like Hibiclens or Bactoshield, etc). Rinse clean and dry with a soft towel. Always keep the shield assembly right side up when cleaning.

Shields can also be cleaned using a cold sterilization solution - always follow the manufacturer's sterilizing instructions. Do not autoclave shields, or visors.

## LIGHT CURING FILTERS

### FOR DENTAL PROFESSIONALS ONLY

The light curing filter protects your eyes during light curing procedures. It is NOT designed or intended for use with lasers.

Peel both sides of the filter and attach it to the inside of the face shield - place it just below your line of sight and use like a bifocal.



### TIME SAVING TIP:

Keep the light curing filter on an extra face shield and use exclusively during light curing procedures.