

THEY'LL NEVER
KNOW THE
DIFFERENCE!



PermaFlo™ Pink

Gingival Replicator

PermaFlo Pink gingival replicator offers an attractive solution for cases in which gingival grafting isn't an option.

- Masks prevalent gingival recession in Class V restorations
- Improves patient comfort by covering exposed root structure



963 - PermaFlo Pink Mini Kit
 2 x 1.2 ml syringes
 4 x Micro 20 ga tips

PermaFlo™ Pink

Instructions:

1. Remove Luer Lock cap and attach a disposable **Micro 20 ga tip** securely onto syringe.
2. Isolate preparation with **Ultrapak™ cord** and/or **DermaDam™ dental dam** to control sulcular fluid.
3. **Etch** 15 seconds.
4. Apply dentin bonding agent (we recommend **Peak™ Universal Bond adhesive**). Air thin, then light cure 10 seconds with VALO™ curing light.
5. Apply first layer of **PermaFlo Pink material** 0.5–1 mm and light cure. Follow with 2 mm increments as needed. Light cure each resin layer 20 seconds. See Fig. 1.
6. Shape with appropriate finishing burs. Finish with strips, polishing cups, and wheels or disks as appropriate. We recommend **Jiffy™ polishing system** for shine, ending with **Jiffy™ HiShine™ polisher**. See Fig. 2.

(Note: Please refer to Permaflo Pink Instructions For Use for complete instructions and precautions)

Fig. 1



Following tooth preparation and application of dentin bonding agent (we recommend Peak Universal Bond adhesive), restoration is built incrementally with PermaFlo Pink material.

Fig. 2



Resultant restoration can mask white root surfaces in cases of recession when gingival grafting isn't an option.

Ultradent Products Used with PermaFlo Pink:



1252 Micro 20 ga Tips

Standard flowable delivery tip easily expresses PermaFlo Pink material. Includes double threads, which lock tip into place for increased security.



4551 Peak™ Universal Bond

The universal formulation of Peak Universal Bond adhesive works with all bonding substrates and provides proven superior bond strengths.



130 Ultrapak™ Knitted Cord Kit

The unique knitted design exerts a gentle, continuous outward force as knitted loops seek to open. Ultrapak cord compresses upon packing then expands for tissue displacement.



848 Jiffy™ Polisher Variety Pk

Three grits are all available in cup, disk, or point form. Excellent for contouring, finishing, and polishing composites such as PermaFlo Pink material.



311 DermaDam™ Medium

The most pure latex rubber dam available. DermaDam dental dam has a high tear resistance and is powder free, which eliminates allergic reactions to powder.



3061 Jiffy HiShine™ Cups

Impregnated with diamonds and silicon carbide to bring a beautiful finishing shine to composites such as PermaFlo Pink material.



383 Ultra-Etch™ IndiSpense™ Kit

The industry leader for over 30 years. Ultra-Etch etchant is viscous enough to prevent migration on a vertical surface while penetrating into the smallest pits and fissures, due to physical and chemical properties that promote capillary action.

